

Silhouette™ Whole Leg and Lower Leg Undersleeve Standard Sizing Chart


Date _____

Patient Name _____



Circumference Measurements			Small	Medium	Large	X-Large
Position	Left	Right	centimeters	centimeters	centimeters	centimeters
Arch circumference A			16 - 23	20 - 30	20 - 30	20 - 30
“Y” circumference B			0 - 30	0 - 36	0 - 36	0 - 36
5cm from ankle			15 - 27	20 - 34	25 - 39	30 - 45
15cm from ankle			24 - 31	29 - 39	34 - 44	39 - 49
25cm from ankle			27 - 35	32 - 43	38 - 48	43 - 53
<i>Stop here for lower legging</i>						
35cm from ankle			31 - 39	36 - 46	41 - 51	46 - 57
45cm from ankle			36 - 43	41 - 50	46 - 55	51 - 60
55cm from ankle			39 - 47	45 - 55	50 - 60	55 - 65

(rev. 0109) ©2009 CircAid Medical Products, Inc. All rights reserved.

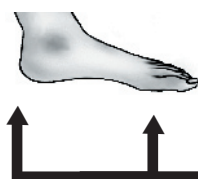

Length from base of toes to back of heel (fits 15 - 25cm)	Left	Right

Lateral length from prominent point of malleolus to gluteal fold	Left	Right

Medial length from prominent point of malleolus to groin, far enough below pubic area so that garment will not cause discomfort (fits 65 - 70cm) <i>(Whole Leg Only)</i>	Left	Right

Medial length from prominent point of malleolus to knee crease (fits 34 - 39cm) <i>(Whole Leg & Lower Undersleeves)</i>	Left	Right

If borderline between sizes, consider amount of edema present and anticipated reduction. If significant edema is present & reduction is anticipated, select smaller size. If edema is minimal, select larger size. Also consider desired fit. Measurements at maximum size range will return a snug fit, while measurements at minimum size will yield a roomier fit. If measurements fall outside parameters in the table, measure for a custom Silhouette legging using the custom legging chart.