




T-3 M™ Legging

Excellent Therapy for Patients That Need a Prescribed Pressure Range
Standard Line Measuring and Sizing chart

(Also used to measure & size T-3 EZ™, C-3 EZ™, T-3 Boot™, & C-3 Wrap™)

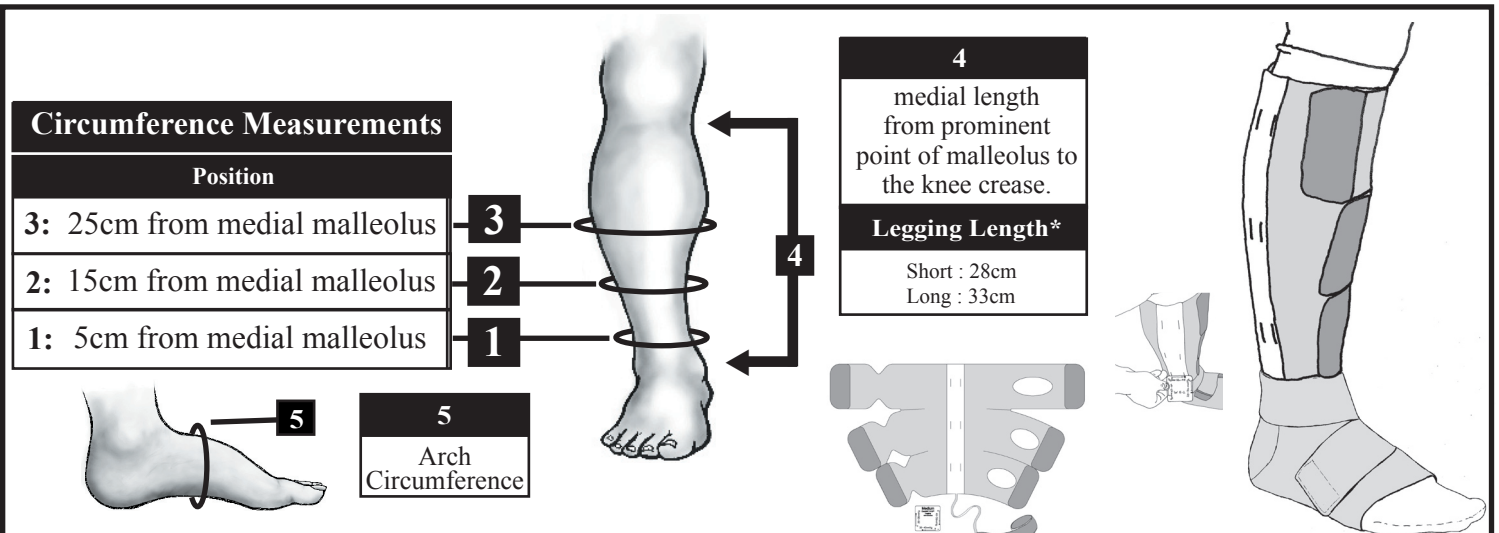

Patient:

- A. Should be standing.
- B. Limb should be at its smallest and limb size at time of measurements should be maintained to assure a proper fit of garment.
- C. Be sure to select a length (#4), legging size (#'s 1-3), and foot size (#4).

Sizing Rules

Measure accurately. If borderline between sizes, consider amount of edema present & anticipated reduction. If significant edema is present and reduction is anticipated, select smaller size; if edema is minimal, select larger size. If measurements do **NOT** fall within **any [one] size category**, a different product type (Juxta-Fit™) will need to be ordered.

	Measurements		Small		Medium		Large		XL	
	Left	Right								
1 5cm			19 - 22cm		22 - 27cm		25 - 32cm		30 - 37cm	
2 15cm			21 - 29cm		28 - 36cm		34 - 42cm		40 - 48cm	
3 25cm			25 - 36cm		33 - 44cm		41 - 51cm		48 - 58cm	
4 *Length			Under 36cm	Over 36cm	Under 36cm	Over 36cm	Under 36cm	Over 36cm	Over 36cm	
			Short	Long	Short	Long	Short	Long	Long	
5 Arch Circumference (Foot Size)			19-26cm	24-31cm	19-26cm	24-31cm	19-26cm	24-31cm	19-26cm	24-31cm
			Small	Med	Small	Med	Small	Med	Small	Med

Foot Piece (Ankle-Foot Wrap)

The T-3 M™ features an attached closed heel Juxta-Lite™ ankle-foot piece (AFW). The foot piece can accommodate the circumference ranges listed above (#5), and a heel-to-base-of toe length of 16cm or greater. If foot measurements fall outside of these ranges and patient fits into a standard size legging please contact us for additional assistance with your order.